


Ocutech VES[®]-K

(Vision Enhancing System)

Fitting Guide

1. The frame and telescope mounting


The frame holds the telescope as well as the eyeglass prescription (if needed) called the “carrier lens.” The telescope is attached to the frame by way of a bridge mounting assembly held in place by two set screws. When loosened, these screws allow the telescope to be slid right and left to align the telescope eyepiece properly for the user’s eye, and also to set the proper angle of the telescope. The telescope eyepiece protrudes through the carrier lens for the eye that will use the telescope and is positioned above the normal line of sight of the eye.

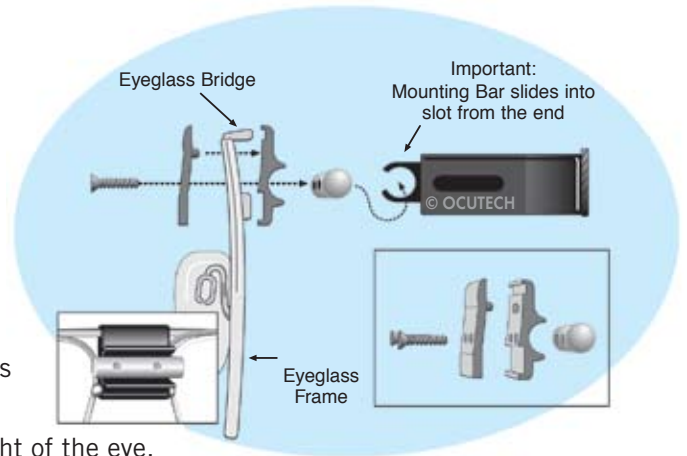


IMPORTANT: *The telescope DOES NOT clip-off the frame. The frame mounting assembly provides a limited range of tilt angle. DO NOT force the angle beyond its free range of adjustment. THIS MAY DAMAGE the telescope and result in a repair.*

2. Positioning the VES[®]-K


A. *Loosen the set screws on the back of the bridge to slide the VES[®]-K right and left to position the eyepiece and to change the angle of inclination of the telescope. You can also slide the telescope completely off the frame, flip it over, and reposition for use by the other eye.*

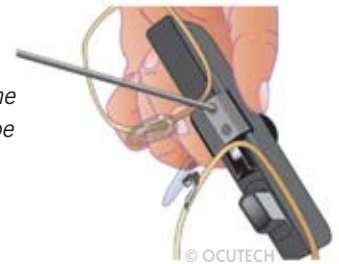


B. *Adjust the upward angle of the telescope to be approximately 10 degrees up from the horizontal.*

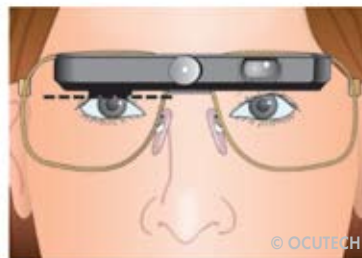

C. *Adjust the bridge so the bottom of the eyepiece is positioned at the top of the pupil or the bottom of the upper lid – whichever is lower.*

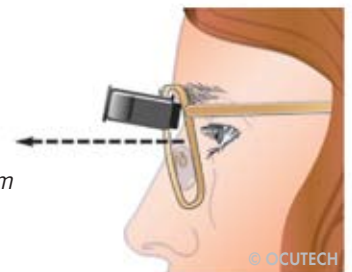

D. *When the patient drops their head down to site through the VES[®]-K there should be a straight line from the eye through the telescope.*

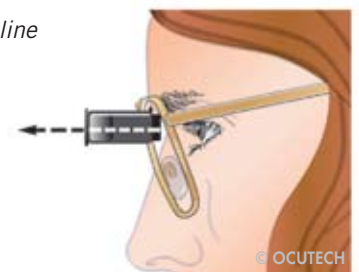

If you need more tilt than that provided by the mounting assembly you can:

1. Either reduce the pantoscopic angle of the frame,
2. Widen the nose pads to lower the frame on the nose,
3. Or place an upward bend on the temple tip as it goes over the ear

Ocutech VES®-K Fitting Guide

3. Adjust the temples

and attach a headband on the temples and tighten gently behind the patient's head to keep the VES® firmly in place.

4. Review the “Patient Instruction Guide”

with the patient. This will help you to teach the patient to sight through the eyepiece, switch between the carrier lens and eyepiece, localize objects with the carrier lens and then switch into the eyepiece, and focus the device.

5. Go over safe handling

cleaning and practice instructions.

6. Remind the patient to

**NOT TURN THE FOCUS KNOB BEYOND THE POINT OF TENSION.
THIS MAY STRIP THE INTERNAL KNOB GEARS.**

OCUTECH® *Vision Enhancing Systems*

109 Conner Drive, Suite 2105, Chapel Hill, NC 27514 USA

+1 (800) 326-6460 • +1 (919) 967-6460 • Fax: +1 (919) 967-8146

Info@ocutech.com • www.ocutech.com



Obelis s.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2. 732.59.54
Fax: +(32) 2.732.60.03
E Mail : mail@obelis.net

Effective August 2010